



TUTORIAL TECHNIQUE FOR HYDRATION OF ALLOOSS™ BLOCK GRAFT

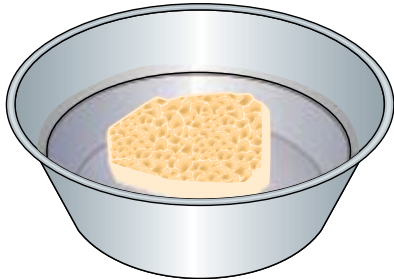

- Place block into a bowl with sterile saline for 30 minutes

OPTIONAL SYRINGE TECHNIQUE FOR QUICK HYDRATION OF BLOCK

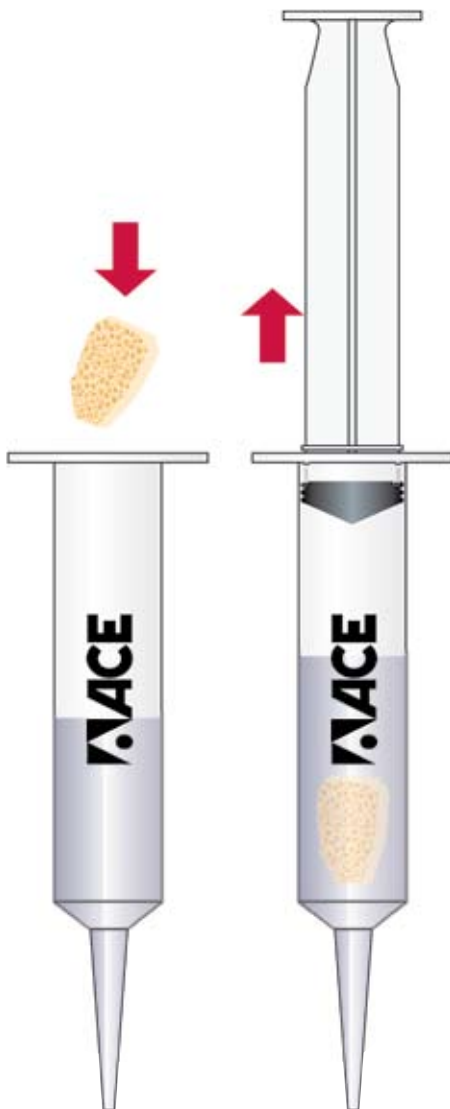

(Prep Time 3-5 minutes)

- Fill a 20cc syringe (003-4302) halfway with .9% Sodium Chloride (Sterile Saline)
- Place the Allograft bone block into the syringe
- Purge the air out of the syringe
- Using your finger hold the tip of the syringe while creating pressure with the plunger for approximately 3-5 minutes
- Remove the block from the syringe and place it into a bowl of Sodium Chloride (until ready to use)